



## QUICK GUIDE

### Unfolding the eFOLDi MK1.5 Scooter

eFOLDi 

Before starting stand the scooter upright in an open space away from pets and small children.

Unfolding the scooter is straightforward, with a little practice you will be able to set it up in seconds.

We advise you to practise with the scooter before venturing out for the first time.



1 Locate 'B' and 'C' latches.



2 Press and hold 'B' and 'C' latches and...



3 ...pull the drag/carry handle upright.



4 Release the handlebar by pulling towards you.



5 While pushing the handlebar down and inwards release the 'E' latch.



6 Pull the steering column up and over the carry handle until it **locks** in place.



7 Release the front wheel by a sharp pull or twisting.



8 Lower the scooter to the ground holding carefully.



9 Pull the steering column upright...



10 ....and make sure the column key **locks** into place under latch 'A'.



11 Pull the seat back upright.



12 Push the footpegs down. Your scooter is now ready.

## Folding the eFOLDi MK1.5 Scooter



**1** Fold the foot pegs to an upright position.



**2** Fold the seat back down.



**3** Press and release latch 'A'.



**4** Fold the steering column flat against the seat.



**5** Lift the scooter to an upright position.



**6** Make sure the front wheel is flat and locked to the frame by its magnet.



**7** Locate latch 'D'.



**8** While supporting the steering column press latch 'D'...



**9** ...and bring the steering column over the carry handle.



**10** Press the steering column in to the frame until latch 'E' locks.



**11** Press handlebars against the frame until the magnet locks them in place.



**12** Press and hold latches 'B' and 'C' and fold the carry handle down.

## The eFOLDi MK1.5 Battery



The battery has points for charging and for powering a mobile device.



Remove the battery by unlocking using the key and applying gentle pressure.



**ALWAYS recharge battery every 3 mths and/or gauge shows 24 or less.**