

Guidance for using the eFOLDi from Transport for London

Wheelchairs and mobility scooters

You can use wheelchairs and some mobility scooters on many services, including buses, Tubes, trains and trams and some boats including Thames Clippers.

Mobility scooters can't be taken on some boats, taxis or the Emirates Air Line.
On almost all buses, the wheelchair ramp is located at the exit door (in the middle of the bus).

Mobility Aid Recognition scheme

This scheme is a way to let bus drivers' that your mobility aid - for example, a mobility scooter has been approved and to automatically let down the ramp for you.

When you apply for a card, a travel mentor will confirm your mobility aid is suited to bus travel and will give you guidance to ensure you're able to safely manoeuvre it to get on and off the bus.

You will also be offered the opportunity to have an accompanied journey, and they will also provide useful tips for how to get safely on and off the bus and plan your journeys.

You can apply for the card if you use a mobility scooter, walking aid or wheelchair buggy. Please note not all mobility aids are suitable for travel on public transport.

If your mobility aid is suited to bus travel, you will be given a Mobility Aid Recognition scheme card which you can show to bus drivers.

For more information go to <https://tfl.gov.uk/transport-accessibility/wheelchair-access-and-avoiding-stairs>. To join the scheme, or to find out more, email travelmentor@tfl.gov.uk or call TfL on 020 3054 4361 between 09:00 - 16:00 Monday to Friday to book an appointment

eFOLDi MK1.5 and the Mobility Aid Recognition Scheme

Will not be accepted by TfL on buses in its unfolded (scooter state). The unfolded scooter is larger than TfL's specified maximum size and TfL do not allow scooters with a potential maximum speed of 8mph to be used on TfL vehicles such as trains and buses. The MK1.5 is acceptable without assessment when folded up and used as mobile luggage.

eFOLDi Lite and the Mobility Aid Recognition Scheme

May be accepted on an individual assessment basis. The user would need to contact TfL's Mentoring service and request an assessment. The Lite will be accepted without assessment when folded and used as rolling luggage.

eFOLDi Power Chair and the Mobility Aid Recognition Scheme

All wheelchairs, both powered and unpowered, are acceptable to TfL without requiring assessment. All busses and trains will accept a wheelchair for transport and no Mobility Aid card is needed.

General Requirements

In order for a mobility device to be generically approved by TfL with no assessments required it must meet the following specifications UNLESS it is a wheelchair type device.

Maximum Width:	600mm
Maximum Length:	1000mm
Maximum Turning Radius:	1200mm