



QUICK GUIDE

Unfolding the eFOLDi Lite Scooter

eFOLDi 


Before starting stand the scooter upright in an open space away from pets and small children.

Unfolding the scooter is straightforward and with a little practice you will be able to set it up in seconds.

We advise you to practise with the scooter before venturing out for the first time.



1 Release steering column latch..

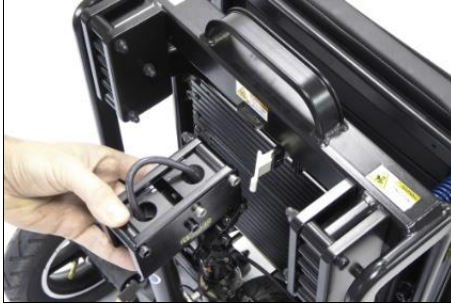

2 ..and allow the column to swing free.



3 Pull the handlebars out and upward..



4 ..until they lock into a vertical line.



5 Locate small black stopper on side and pull to release front wheel assembly.



6 Hold the steering column and seat back and pull them apart..



7 ..the scooter will unfold as you pull the seat back and steering column apart..



8 Pull apart and..



9 ..the scooter will lock itself into its open position.



10 Make sure the locking latches have locked to the steel bar..



11 ..if not press your foot down onto the footplate to lock the scooter open.




12 Check that the magnetic brake is in the **LOCKED** position to engage the motor.

Folding the eFOLDi Lite Scooter



1 Fold the seat backrest down to the seat base.

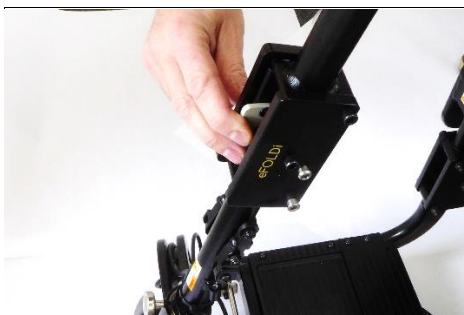

2 Press and hold the small latch on the steering column.



3 ...and fold the column down until.



4 ...it is locked by the magnetic catch.



5 Firmly hold the scooter by its steering column and the carry handle and firmly press the column down towards the footplate



6 Tip the scooter back at an angle and use your foot behind the wheel to stop it moving away from you



7 ...press inwards on the steering column until the front wheel locks into place.



8 ...and steering column latch locks into position.



9 Press in steering column to lock it tightly against the frame.

eFOLDi Lite Operation



The battery can be removed by lifting the latch and lifting upwards.
There is a charging point on the side of the battery.



To turn the scooter on press the grey button below the battery display.
Select (D) Drive or (R) Reverse and press the thumb throttle to go.

Always recharge the battery as soon as possible when the display is indicating amber.



To turn the scooter into rolling luggage ensure the vertical steering column is locked against the steering column latch.