


eFOLDi[®] Explorer Quick Start Guide



Before using, position the eFOLDi Explorer in an open area away from pets and small children. Unfolding is easy, and with practice, you can set it up in seconds. We recommend practicing before your first outing.

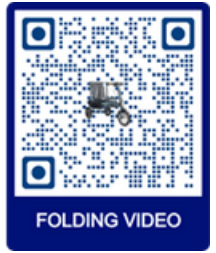

HOW TO UNFOLD


STEP 1 - ASSEMBLE INTO ROLLING LUGGAGE

Remove all packaging before you begin. The charger is located in the polystyrene liner and the keys are in the Explorer seat back bag.



Locate the "B" & "C" latches. Press them and pull the drag/carry handle into its upright position.

STEP 2 - RELEASE THE STEERING COLUMN

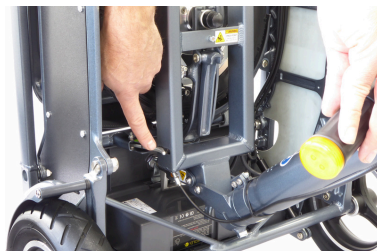

Release the handlebar by pulling towards you.

While pushing the handlebar down and inwards, release the "E" latch.

Pull the steering column up and over the carry handle until it locks into place.

Release the front wheel with a sharp pull or twist.

STEP 3 - UNFOLD AND LOCK



Pull the wheel to the front position and lower the scooter carefully to the ground.

Pull the steering column upright and make sure the column key locks into place under latch "A".

Pull the seat upright and push the foot pegs down.

GETTING STARTED

HOW TO OPERATE



To turn on the scooter, insert the key and turn clockwise from the 9 to 11 o'clock position. The twist throttle is on the right handle. The brake is manual and located alongside the left handle. The battery level is shown on the display bar. We recommend charging before the display reaches the last bar — once it shows one bar, the battery is fully empty."



To charge, connect the power supply cable to the charging port located on the battery. There is also a USB port for charging and powering a mobile phone.



To remove the battery, unlock using the key and lift gently upwards.

*Keys found in grey seat bag

HOW TO FOLD AND STORE

STEP 1 - FOLD THE BACK REST & STEERING COLUMN



Fold the seat back down and the foot pegs into an upright position.



Press and release latch "A", folding the steering column flat against the seat.



Lift the scooter to an upright position and lock the front wheel flat onto its magnet in the frame.




Locate latch "D" and press while supporting the steering column.

STEP 2 - COLLAPSE & LOCK THE SCOOTER



Lower the steering column over the carry handle.

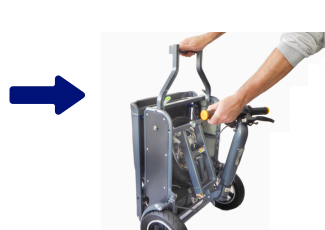

Press the steering column into the frame until latch "E" locks.



Press the handlebars against the frame until the magnet locks them into place.



Press and hold latches "B" and "C" and fold the carry handle down.

Please note this leaflet is intended to be a quick guide only. For full instructions, please refer to our user product manual. If you have any questions, please call us on 0800 915 0800 or email services@efoldi.com