

eFOLDi[®] Navigator Quick Start Guide



Before using, position the eFOLDi Navigator in an open area away from pets and small children. Unfolding is easy, and with practice, you can set it up in seconds. We recommend practicing before your first outing.



HOW TO UNFOLD



Your eFOLDi Navigator has an accessory bag. Inside you will find the anti-tip wheels, a battery charger, and optional handles.



Lay the outer carton flat and remove all packaging from the Navigator before assembly.



Stand the chair up and undo the seatbelt clip securing the chair in its folded position.



Unfold the chair by pushing the seat down.



Insert the anti-tip wheels into the back of the chair frame with the sprung buttons facing inward.



Install the arms by pushing them down until you hear a click and the red button locks.



Insert the joystick rod under the armrest and tighten the knob very securely.



Install attendant handles by screwing them into the top of the chair frame.



Attach attendant control to the top of the chair frame.



Connect the power cable to the battery by sliding the port cover & inserting power cable.



Push the foot rest down.



Place the non-slip cushion onto the seat (bobbly side down), securing with the Velcro tabs.



The red brake levers are set in the forward position which engages the motor. For free-wheel mode, pull both levers into the backward position.



Turn on the power button, refer to the user manual for more information.

READY TO ROLL




The handset controller features buttons for adjusting speed, with 5 power levels and 5 speed settings (1 = low, 5 = high).

To disable the reversing sound, simply press the button (shown above).



The attendant control adjusts walking speed for your assistant and provides a battery check, with a lever for forward and backward motion.

BATTERY CHARGING

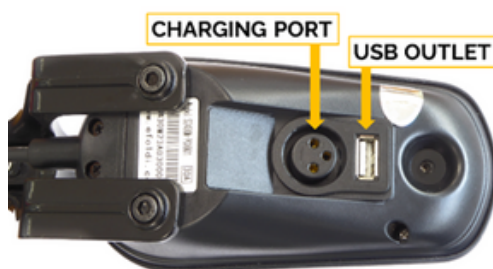

The charging port is under the handset, and there's also a USB outlet for powering devices.



To charge the eFOLDi Navigator, plug the charger into the joystick base and connect the AC plug to a power outlet. Alternatively, you can remove the battery and charge it separately - leaving the chair in your car if required.

HOW TO FOLD TO STORE



Use the POWER button on the hand controller to turn off the chair when not in use to conserve battery power.



Fold the footrest up.



Press the folding lever (located under the front edge of the seat) & lift the seat to fold backwards.



Sit the chair back onto the anti-tip wheels to store.



Please note this leaflet is intended to be a quick guide only. For full instructions, please refer to our user product manual. If you have any questions, please call us on 0800 915 0800 or email hello@efoldi.com