

## Before You Travel

- ☐ Contact your airline early to confirm acceptance of your eFOLDi scooter or power chair and its battery
- ☐ Check the latest CAA and IATA guidance for mobility devices with lithium-ion batteries
- ☐ Familiarise yourself with your eFOLDi model – know how to fully switch it off and remove the battery
- ☐ Go to [www.efoldi.com/guides-and-manuals](http://www.efoldi.com/guides-and-manuals) to download your requirements:
  - ☐ eFOLDi owner's manual – it will help you complete your travel pack form
  - ☐ eFOLDi Travel Pack form to organise the information airlines typically require
  - ☐ eFOLDi model's UN38.3 test summary for your eFOLDi lithium-ion battery

## At the Airport

- ☐ Remove accessories (such as baskets or cushions) and pack them separately
- ☐ Help protect vulnerable components during handling – fit your eFOLDi protection case if you have one
- ☐ Provide full eFOLDi scooter or chair details (model, dimensions, battery type) when arranging special assistance
- ☐ Arrive in good time – 2 hours for short-haul, 3 hours for long-haul flights
- ☐ Be prepared to demonstrate how your eFOLDi is switched off and the battery isolated/removed
- ☐ Keep the following accessible:
  - UN 38.3 test summary
  - eFOLDi owner's manual
  - Scooter or chair dimensions
  - Emergency contact details
- ☐ Consider attaching clear, high-visibility labels stating:  
“Lift from frame only – NOT armrests”

## On Arrival

- ☐ Inspect your eFOLDi scooter or power chair immediately on return
- ☐ Before using, check fastenings, controls and tyres to ensure everything is secure

**This is a guide only:** Requirements vary significantly between airlines and countries. Always check with your specific airline before booking and travelling. Airlines have the final authority on whether passengers and equipment can fly.