



Before using, position the eFOLDi Cliq in an open area away from pets and small children. Unfolding is easy, and you can set it up in seconds. We recommend practicing before your first outing.



CLIQ TOGETHER

STEP 1 – ASSEMBLE THE STEERING COLUMN

Remove all packaging before you begin.



Join the tiller to the scooter:
unscrew & remove the cog, slot the tiller onto the pin, and replace & secure the cog without tightening it – leave a gap in the teeth.

Lift the steering column into the upright position.

Secure the steering column by twisting the cog (clockwise). You can position the steering column to suit your reach & comfort.

STEP 2 – JOIN THE SCOOTER



Align the main body of the scooter with the rear section.



Place the hooks over the metal bar, and press down until they click into place



Make sure the hooks are secured.

STEP 3 – ADD FINAL PARTS & ACCESSORIES



Insert the seat post



Attach the chair to the scooter



Insert the battery



Attach the basket to the tiller

GETTING STARTED



To turn on the scooter, turn the key clock wise. Adjust speed controller to a speed you are comfortable using.

HOW TO OPERATE



To charge, connect the power supply cable to the charging port located at the top of the steering column.



To remove the battery, lift the latch and carefully pull it upwards. The charging port is located on the side of the battery.

TAKE APART

STEP 1 - REMOVE PARTS & ACCESSORIES



Remove the basket



Remove the seat



Remove the battery

STEP 2 - TAKING APART THE SCOOTER



Lift grey handle to separate back section



Turn the tiller cog to lower tiller until it is laying flat

