

SUN0263 Footrest Extension Bundle

Fitting Guide



The eFOLDi Footrest Extension Bundle is suitable for any eFOLDi Lite with a serial commencing LT202206 or higher.

The bundle comprises two footplate extensions and the Philips headed bolts for securing them to the eFOLDi Lite Frame.



Check that the Lite scooter has the factory drilled and tapped holes on either side of the frame as illustrated.

Scooters without the factory drilled and tapped holes are not suitable for conversion due to the extra strengthening of the frame required.



Align the footrest extension to the threaded holes on the scooter.



Fit the Philips headed bolt into the threaded hold closest to rear of scooter and screw in. Do not tighten until the second bolt is located.



Fit the bolt nearest the front of the scooter and tighten down and then tighten the rearmost bolt from the previous step.

Follow the same procedure for the extension on the other side of the scooter.

This completes installation of the SUN0263 Footrest Extension Bundle