

eFOLDi[®] Lifter Quick Start Guide

Before using the eFOLDi Lifter make sure you have a level surface and remember to balance the scooter so that it is stable and centred on the Lifter platform when in use.



HOW TO LOAD

STEP 1 - SET UP



Lay the blanket inside the car boot and allow part of it to drape over the edge.



Position the lifter on flat ground. The long side should face the car.



Place the lifter about 1 inch (2.5 cm) away from the back of the vehicle.



Remove the scooter battery, then plug the battery connector into the lifter.

STEP 2 - LOADING YOUR SCOOTER



Place the scooter onto the lifter. Make sure it is centred and balanced, with the seat facing the vehicle.



Use the handset to raise the lifter platform, keeping one hand on the scooter to keep it stable and balanced.



Make sure the lifter platform lines up with the boot edge before you start loading.



Fold out a section of the blanket to protect the vehicle's paintwork.

STEP 3 - TRANSFERRING YOUR SCOOTER



Carefully roll the scooter over the gap and into the vehicle.



Place the scooter in the desired position inside the boot, ensuring it is stable.



Fold the blanket back into the boot. Disconnect the battery and store the lifter. **Keep the blanket away from the boot catch.**

STEP 1 - SET UP



Position the lifter on flat ground. The long side should face the car.



Position the lifter about 1 inch (2.5 cm) away from the rear of the vehicle.



Connect the battery connector to the battery.



Use handset to raise the Lifter platform.

STEP 2 - TRANSFERRING YOUR SCOOTER



Make sure the lifter platform lines up with the boot edge.



Fold out a section of the blanket to protect the vehicle's paintwork.



Roll the scooter across the gap onto Lifter platform.



Place the scooter upright and make sure it is centred and balanced safely.

STEP 3 - UNLOADING YOUR SCOOTER



Fold the blanket back into the vehicle, keeping it clear of the boot catch **BEFORE** lowering the lifter.



Use the handset to lower the lifter platform, making sure the scooter stays stable and balanced.



When Lifter is at it's lowest position, lift the scooter off.



Disconnect battery and replace in scooter. Pack Lifter away safely.